



Association for
Clinical Cytogenetics

TAG **CMGS** CGTGTCAGTACCGTACTTAGC
ACT ACGTGTCAGTACCGTACTCGT
TACCGTACTTAG **CLINICAL MOLECULAR** ACGTG
CAGTACCGTACT **GENETICS SOCIETY** ACGTGTC/

PROFESSIONAL GUIDELINES FOR CLINICAL CYTOGENETICS AND CLINICAL MOLECULAR GENETICS

QF-PCR FOR THE DIAGNOSIS OF ANEUPLOIDY BEST PRACTICE GUIDELINES (2012) v3.01

January 2012

CONTENTS

1	INTRODUCTION	3
2	SERVICE OVERVIEW	3
3	SAMPLE PREPARATION	4
3.1	Amniotic Fluid Samples (AF)	4
3.2	Chorionic Villus Samples (CVS)	4
3.3	Maternal Blood Samples	4
3.4	Neonatal Blood Samples	5
4	DNA EXTRACTION	5
5	PCR REACTION	5
5.1	Array Design	5
5.2	PCR Programme	6
6	ANALYSIS	6
6.1	Genetic Analysers	6
6.2	Interpretation of QF-PCR Products	6
6.2.1	Interpreting Allele Ratios	7
6.2.2	Interpreting Assay Results	8
7	MATERNAL CELL CONTAMINATION	10
7.1	Blood stained Amniotic Fluid Sample	10
7.1.1	Low Level Maternal Cell Contamination	10
7.1.2	Single Genotype Present	10
7.1.3	Inconclusive Allele Ratios	10
7.2	Chorionic Villus Samples	11
8	MOSAICISM	11
9	REPORTING	12
9.1	Normal Results	12
9.2	Reports With Results Indicating Autosomal Aneuploidy	12
9.3	Reports Indicating Sex Chromosome Monosomy	13
10	REFERENCES	13
11	VERSION CONTROL	15

1 INTRODUCTION

These guidelines should be used in conjunction with all other Association for Clinical Cytogenetics Best Practice Guidelines.

Professional guidelines for cytogenetics laboratories incorporate the standards imposed by regulatory bodies (Clinical Pathology Accreditation (CPA) [1] and by statute (Clinical Governance) while taking into account current practice in the U.K.

Elements of the service not subject to statute may be varied in order to comply with local constraints and agreements. It must be noted that these guidelines are minimum requirements and that professional judgement is of paramount importance for many circumstances.

The use of 'shall' in this document indicates a requirement and the use of 'should' indicates a recommendation. In addition the use of 'acceptable' highlights an area where more than one approach is satisfactory and 'unacceptable' indicates areas where the quality of the service may be compromised. Where there appears to be a contradiction between available guidelines the most recently published should be taken to apply to all. The use of QF-PCR analysis of short tandem repeats (STR) for the detection of aneuploidy has since been validated and successfully applied for the rapid diagnosis of prenatal aneuploidy by a number of UK and European labs. The rapid result may be followed by full karyotype analysis of cultured cells.

2 SERVICE OVERVIEW

The testing of prenatal samples may be complicated by limited sample quantity, variable sample quality, mosaicism and maternal cell contamination. The processing of a number of prenatal samples requires stringent quality control measures to minimize the risk of sample mix-up. These aspects of prenatal diagnosis are taken into account in these guidelines.

Both cytogenetic and molecular genetic support should be sought for validation and implementation of a rapid aneuploidy diagnostic service by QF-PCR. Testing for trisomies 13, 18 and 21 should be routinely carried out. It is acceptable to test for sex chromosome aneuploidy in only a subset of referrals indicating a likely sex chromosome abnormality; the extent of sex chromosome testing carried out is decided at a local level in consultation with clinicians and other relevant groups.

In line with the National Screening Committee guidelines, 97% of samples should receive reports within 3 working days following sample receipt (Antenatal Screening – Working Standards for Down's syndrome screening (2007), section 14

<http://fetalanomaly.screening.nhs.uk/standardsandpolicies>)

3 SAMPLE PREPARATION

As cell cultures may need to be established for karyotype analysis, only a proportion of the sample may be available for QF-PCR aneuploidy testing. Sample preparation (cleaning of chorionic villus samples and aliquoting of amniotic fluid samples) should be carried out in accordance with good laboratory practice including no pre-labelling of vessels and the processing of only one sample at any one time.

Protocols should minimise the risk of sample mix-up. All tube-tube transfers should be checked by a second individual and the identity of samples shown to have an abnormal result should be confirmed before results are issued. This can be done in a number of ways, including analysis of a maternal DNA sample to confirm sample identity or a second test on the original sample.

3.1 Amniotic Fluid Samples (AF)

Generally between 0.5 and 4 ml (or 1/10 of the sample) is used for QF-PCR analysis, though it is noted that the larger aliquot sizes suggested may compromise subsequent karyotype analysis, particularly for early gestation/small samples. The size of the aliquots taken routinely from samples is a local decision.

3.2 Chorionic Villus Samples (CVS)

For chorionic villus samples (CVS), an aliquot of dissociated villi (either digested or finely chopped) representing different regions of the sample should be processed in order to minimise the risk of misdiagnosis due to confined placental mosaicism (see Mosaicism section 8 below). Such dissociated samples are likely to represent both cytotrophoblast and mesodermal tissue. If the sample is small, or of poor quality, then processing a single villus is acceptable, however, it should be noted that mosaicism may not be detected and confirmation of fetal origin may be required.

3.3 Maternal Blood Samples

Maternal blood samples can be used to determine the fetal genotype in cases of maternal cell contamination or poor sample quality in a prenatal sample, to confirm the identity of samples with abnormal results and occasionally, to predict the clinical significance of anomalous results. It is a local decision whether maternal blood samples are requested with all prenatal samples or for individual cases.

3.4 Neonatal Blood Samples

Neonatal blood samples referred for trisomy 13, 18, 21 and sex chromosome testing can be tested by QF-PCR to provide a rapid result. The referral types for which testing is offered is a local decision.

4 DNA EXTRACTION

Any method must be fast, require a minimal number of tube-tube transfers and produce a standard quality of DNA, even from small samples, that is reliable for use in PCR multiplex assays.

5 PCR REACTION

Commercial QF-PCR assays or commercial buffers suitable for multiplexing several primers are used for the PCR amplification. Laboratories using in house assays may prepare kits in advance by the addition of primers and H₂O aliquoted into individual tubes. Batch testing of kits should be carried out to ensure both consistent assay quality and trisomic and normal control sequences are accurately represented.

A H₂O control must be included in each PCR set-up to identify any DNA or PCR product contamination. It is acceptable to use DNA controls (trisomy or normal) with each PCR run in order to monitor variation between runs, but these are not necessary to validate the dosage results; the dynamics of an individual PCR will vary due to differences in DNA concentration/contaminants and therefore the quality of each result should be assessed independently (see analysis). Post-PCR clean-up is not necessary.

5.1 Array Design

A minimum of 4 markers should be tested for each autosome and X chromosome. The use of more markers may provide additional information, particularly in cases with abnormal results. A minimum of two Y specific sequences is recommended to determine sex chromosome status which should include AMEL (Sullivan, 1993) or similar to determine the ratio of X to Y sequences. In addition the use of an X chromosome counting marker is recommended to determine the number of X chromosomes in the absence of a Y chromosome; the TAF9 sequences (Deutsch 2004) present on 3p24.2 and Xq21.1 are amplified using the same pair of primers. They can be separated according to length and then compared to assess X chromosome copy number.

Basic principles of multiplex design should be followed in designing assays, including primers > 22 bp in length and avoidance of homology to repetitive DNA, SNPs and primer-dimer interactions by sequence comparison. Tri/tetra/penta/hexanucleotide repeat markers should be used as these have fewer stutter peaks, although it is acceptable to use dinucleotide repeat markers if few suitable markers are available within the tested region. All markers should have high heterozygosity within the population being tested. New markers not used previously for PCR aneuploidy diagnosis should be validated by testing a minimum of 100 chromosomes, including aneuploid samples to confirm marker location and to exclude common polymorphisms.

5.2 PCR Programme

To ensure the reaction remains in the semi-quantitative phase, 24-26 PCR cycles should be carried out as standard practice. It is acceptable to use a higher number of PCR cycles if the normal number of cycles consistently gives low amounts of product as long as the assay is validated for that cycle number and both trisomy and normal controls show correct allele dosage ratios for all markers. The annealing temperature should be set as low as is practical to minimise the effect of primer site polymorphisms.

6 ANALYSIS

6.1 Genetic Analysers

A genetic analyser capable of 2 bp allele resolution and peak area/peak height quantification should be used. ABI 310, 3100, 3130, 3700 and 3730 models (POP4/POP6/POP7/Dmatrix/G5matrix) and Beckman Coulter CEQ8000 (LPA1 gel) analysers have been used by service labs. To ensure the quality of the data, minimum and maximum peak heights in accordance with the genetic analyser's specification must be observed (e.g. for the ABI 3100, markers with peaks above 6000 arbitrary fluorescence units should be excluded from analysis).

6.2 Interpretation of QF-PCR Products

Both the electrophoretogram and peak measurements, which can be transferred to a spreadsheet for convenience, should be used for analysis. It is acceptable to use peak height, peak area or both measurements to calculate allele ratios, although for results obtained from an ABI genetic analyser peak area should be used to minimise the peak distortion caused to widely-spaced alleles. The use of POP4 polymer may result in

reduced allele resolution and in such cases peak height may be of additional use. The area/height of the shorter length allele should be divided by that of the longer length allele to provide the allele ratio. It is acceptable to fail individual markers if valid technical reasons such as electrophoretic spikes or bleed-through between colours are present on the electrophoretogram.

If dinucleotide repeat markers are used care must be taken with analysis of stutter patterns. Dinucleotide repeat markers with alleles separated by 2 bp should not be analysed, as stutter peaks are incorporated into the shorter length allele and may misrepresent allele ratios. For alleles separated by more than 2 bp, it is acceptable to include stutter peaks areas in ratio calculations.

6.2.1 Interpreting Allele Ratios

The ranges outlined below allow for preferential amplification of the shorter length allele, which may be considerable if alleles are widely separated. Occasionally both abnormal and normal allele patterns are obtained for a single chromosome. These results may represent clinically significant partial chromosome imbalance, copy number variants (CNV), primer site polymorphisms (PSPs) or somatic microsatellite mutations (SMMs). The analysis and interpretation of such findings are detailed later in this section.

6.2.1.1 The Normal Range

The normal range for allele ratios should not exceed 0.8-1.4. However, for alleles separated by more than 24 bp an allele ratio of up to 1.5 is acceptable and can be considered to fall within the normal range.

6.2.1.2 The Abnormal (triallelic) Ranges

Three alleles are evident by three peaks in a 1:1:1 ratio or two alleles in 2:1/1:2 ratios. Values between 0.45 and 0.65 and between 1.8 and 2.4 define the triallelic range for alleles where two of the three alleles are of the same size.

6.2.1.3 The Intermediate Ranges

Values falling in the ranges (1.4-1.8 & 0.65-0.8) lying outside the normal and triallelic ranges are referred to as intermediate ranges and the ratio calculations are referred to as inconclusive results. Such results can usually be resolved by re-testing the sample. Follow-up studies are carried out at the discretion of the individual laboratory. Inconclusive results may be caused by PSPs (see below).

6.2.1.4 Primer Site Polymorphisms

Inconclusive results may be caused by PSPs; retesting using a lower PCR annealing temperature should correlate with greater amplification of the PSP allele and a change in the allele ratio. Although a lower annealing temperature may result in a normal allele ratio, markers observed to have a possible PSP should not be used in interpreting the final result, as amplification of the PSP allele may not be complete and may rarely represent an abnormal ratio.

6.2.1.5 Sex Chromosome Markers

The abnormal ranges given above may not apply and may instead represent mosaicism for monosomy X. In addition, sex chromosome markers may give ratios above 2.4 or below 0.45, such ranges indicate the presence of more than 3 copies of the sequence and may correlate with rarer sex chromosome aneuploidies.

6.2.1.6 Somatic Microsatellite Mutations (SMMs)

Somatic microsatellite mutations may be identified by testing a different cell population such as cultured cells and observing altered allele ratios. They arise when a mutation occurs within the microsatellite usually involving an increase or decrease of 1-2 repeats. As all cells should contain one copy of either the original (A) or the mutated (B) allele, the sum of the area/heights for these alleles should give a ratio in the normal range when compared with the unaffected allele (C). When this (A+B)/C pattern gives a ratio in the normal range, the marker result may be failed and not contribute to the result.

6.2.2 Interpreting Assay Results

At least one clinical scientist registered with the Health Professions Council and with appropriate training, should analyse or check each result. Reports should be authorised by an appropriately trained and experienced State Registered Clinical Scientist in line with CPA standards for accredited laboratories. To interpret a result it is important that there is consistency between all markers on the same chromosome. The presence of only one allele for a specific marker indicates likely homozygosity at that locus and is considered to be uninformative.

6.2.2.1 Normal Results

At least two informative marker results consistent with a normal biallelic pattern (with all other markers uninformative) are required to interpret a result as normal. It is acceptable to report cases where only a single marker shows a normal diallelic pattern, all other markers being uninformative, as this is consistent with a normal chromosome complement. The report should state that the result is based on a single marker result

and that this result will be confirmed by another technique (either full karyotype or FISH analysis of uncultured cells).

6.2.2.2 Abnormal Results (non-mosaic trisomy)

To interpret a result as abnormal, at least two informative marker results consistent with a triallelic genotype are required, with all other markers uninformative. It is unacceptable to interpret a result as abnormal if this is shown by only one marker. The presence of three differently sized alleles for one or more markers is consistent with a meiotic nondisjunction event generating the trisomy cell line. The absence of such a result indicates a possible mitotic non-disjunction event, particularly if a larger number of markers are used. Where this pattern is seen in CVS the risk of confined placental mosaicism is increased and therefore this risk should be detailed in the report.

Prior to reporting an abnormal result, sample identity should be confirmed, for example by analysis of a maternal DNA sample or by a second assay (QF-PCR or interphase FISH) on the original sample.

6.2.2.3 Sex Chromosome Status

Although XY, XXY, XYY chromosome complements can be determined by AMEL, and XX and XXX and monosomy X results can be determined by an X chromosome counting marker (e.g. TAF9L), it is important that these markers should not be used in isolation. X chromosome polymorphic markers, Y specific sequences and/or other X chromosome counting markers should also be used as polymorphisms may affect sequence amplification.

6.2.2.4 Single Abnormal Markers

If discrepant abnormal/normal results are obtained for the most proximal or distal chromosome regions this may represent clinically significant regional imbalance. These findings should be detailed in the report and further testing recommended. Abnormal markers that are flanked by normal markers may represent CNV. If the marker has previously been reported to represent a CNV inherited from a normal parent then it is acceptable not to report the abnormal marker result. A collated list of inherited CNV identified by markers used in QF-PCR assays is available from K Mann (kathy.mann@gsts.com).

For markers that have not previously been reported as representing inherited CNV, it is recommended that the abnormal and normal results are detailed in the report. QF-PCR analysis of parental samples may provide information regarding the significance of these results.

7 MATERNAL CELL CONTAMINATION

7.1 Blood stained Amniotic Fluid Sample

It is recommended to process all blood-stained samples, though care must be taken when interpreting female results as the blood may be fetal or maternal in origin. For female samples with high levels of blood staining a maternal genotype is essential for the interpretation of the result: if maternal samples are not available in these cases the PCR result should not be interpreted. Levels of blood-staining in the cell pellet may also correlate with the level of the maternal genotype, though this should be carefully validated before being applied to interpret results.

The detection of maternal cell contamination in the absence of blood staining may indicate the presence of solid maternal tissue. This tissue may be a source of metaphases and care should be taken in the interpretation of karyotype results from these samples.

The presence of two genotypes in DNA from bloodstained samples is consistent with maternal cell contamination. Results from bloodstained samples can be divided into three categories;

7.1.1 Low Level Maternal Cell Contamination

If a low level maternal genotype is present and the majority fetal genotype has no inconclusive allele ratios then the result can be interpreted as described in the Analysis Section 6.

7.1.2 Single Genotype Present

If a single fetal genotype is obtained then the result can be interpreted as described in the Analysis Section.

If required for results from 7.1.1 and 7.1.2, the fetal genotype can be determined by analysis of a maternal DNA sample or by sexing the sample if the fetus is male.

7.1.3 Inconclusive Allele Ratios

If one or more allele ratios are inconclusive (fall between the normal and abnormal range) due to the presence of a second genotype, the fetal genotype should not be interpreted. In cases where a QF-PCR result is not available due to maternal cell contamination of the sample, analysis of cultured cells is usually possible.

7.2 Chorionic Villus Samples

Maternal material may be present in CVS material prepared for QF-PCR analysis if the sample is of poor quality and/or the removal of maternal decidua is incomplete. Maternal cell contamination in CVS of reasonable quality is generally avoidable and may compromise the QF-PCR and/or karyotype results; every effort should be made to consistently remove all maternal decidua from CVS. The fetal genotype can be identified by analysis of a maternal sample or by sexing the sample if the fetus is male. If one or more allele ratio is inconclusive (falls between the normal and abnormal range) due to the presence of a second genotype, the fetal genotype should not be interpreted. Whilst remaining maternal blood in the sample may be a source of maternal cell contamination in CVS, maternal tissue is the most likely source maternal cell contamination and also therefore of metaphases in long-term cultures; care should be taken in the interpretation of karyotype results from samples indicating the presence of maternal cell contamination.

8 MOSAICISM

Mosaicism for trisomy and normal cell lines can be detected by QF-PCR analysis, evident as extra peaks and skewed allele ratios on a chromosome-specific group of markers. These results may be subtle; skewed allele ratios representing the mosaic chromosome may remain in the normal or abnormal ranges (i.e. ratios may all be borderline inconclusive or where the greater area/height is associated with the longer allele).

It is recommended that subtle but consistent skewing and/or small extra peaks on a chromosome-specific group of markers should be investigated further. Usually it is possible to distinguish between a mosaic genotype pattern and a pattern representing two genotypes. If the QF-PCR result can be confidently interpreted the mosaic result should be reported, although the clinical significance of a mosaic result should be carefully considered in the report.

Rare completely discrepant QF-PCR and karyotype results due to placental mosaicism in CVS have been reported (see references below). In most of these cases QF-PCR testing of small or confined regions of the CVS was carried out and the result may represent predominantly cytotrophoblast material. Testing of dissociated cells representing the whole CVS has recently been introduced to the majority of UK labs and is now recommended practice (see Sample Preparation Section 3). The abnormal cell line in most cases exhibited only 2:1/1:2 abnormal results consistent with a mitotic non-disjunction event (see section 6.2.2.2).

9 REPORTING

Reports should state the assumption that fetal material is tested. The fact that the test may not detect mosaicism or small segment imbalance for chromosomes tested should be included on the report, either in the main text or in a report rider. The result may be described as preliminary or stand alone. However, if the QF-PCR test is to be followed by full karyotype analysis, this should be stated.

9.1 Normal Results

It is acceptable to list markers on a normal report, although this should be done in a way that does not 'bury' the result. It is acceptable to report normal QF-PCR results as 'consistent with a normal diploid complement for chromosomes 13, 18 and 21', 'an apparently normal complement of chromosomes 13, 18 and 21 was detected', 'no evidence of trisomy' or similar phrasing.

9.2 Reports With Results Indicating Autosomal Aneuploidy

For abnormal reports, the locations and/or names of markers showing a triallelic result should be listed in the report to define the trisomic region. It is unacceptable to report the QF-PCR result as trisomic for chromosome 13/18/21 as only specific sequences from each chromosome are tested. Report wording stating the result 'indicates' or 'is consistent with' trisomy 13/18/21 is acceptable, as is stating the result is 'trisomic for at least the region represented by' followed by the marker names and locations. Such abnormal reports should also include an interpretative statement such as 'consistent with Down syndrome', 'associated with Down syndrome', 'indicative of Down syndrome' or 'predicted to be affected with Down syndrome'. Trisomy results in CVS samples with no evidence of a meiotic non-disjunction event (see sections 6.2.2.2 and 8) should be reported more cautiously and reports may recommend waiting for the karyotype result, particularly in the absence of ultrasound anomalies. The results for other tested chromosomes should be stated.

Abnormal results that may indicate a familial chromosome rearrangement in one of the parents should be followed by full karyotype analysis of the sample and/or parents to assess recurrence risk.

9.3 Reports Indicating Sex Chromosome Monosomy

It is important to be aware that a QF-PCR sex chromosome assay based on hemi/homozygosity of polymorphic makers is a highly stringent screen for X chromosome aneuploidy but not a diagnostic test. A result consistent with monosomy X, where all polymorphic markers have only a single allele peak and no Y sequences present may represent a normal female homozygous for all markers tested. The risk of this increases with the testing of fewer polymorphic markers and/or if there is consanguinity in the family. In addition an apparent 2:1/1:2 ratio may indicate triple X or 45,X/46,XX mosaicism. X chromosome counting sequences (e.g. TAF9L) are recommended to confirm X chromosome aneuploidies, the use and location of which should be detailed in the report. In the absence of a marker such as TAF9, abnormal results indicating monosomy X or triple X mosaicism in the absence of a triallelic result, should be confirmed using another technique, or the limitations of the test should be stated in the report.

10 REFERENCES

Allen S K, Luharia A, Gould C P, Macdonald F, Larkins S, Davison E V 2006. Rapid prenatal diagnosis of common trisomies: discordant results between QF-PCR analysis and karyotype analysis on long-term culture for a case of Trisomy 18 detected in CVS. *Prenat Diagn* 26(12):1160-7.

Cirigliano V, Ejarque M, Canadas M P, Lloveras E, Plaja A, Perez M M, Fuster C and Egozcue J 2001. Clinical application of multiplex quantitative fluorescent polymerase chain reaction (QF-PCR) for the rapid prenatal detection of common chromosome aneuploidies. *Mol Hum Reprod* 7(10): 1001-6.

Deutsch S, Choudhury U, Merla G, Howald C, Sylvan A, Antonarakis SE 2004. Detection of aneuploidies by paralogous sequence quantification. *J Med Genet* 2004;41:908-915. Donaghue C, Roberts A, Mann K and Ogilvie C M 2003. Development and targeted application of a rapid QF-PCR test for sex chromosome imbalance. *Prenat Diagn* 23(3): 201-10.

Feuk L, Marshall C R, Wintle R F, Scherer S W 2006. Structural variants: changing the landscape of chromosomes and design of disease studies. *Hum Mol Genet* 15 (Review Issue 1): R57-66. Levett L J, Liddle S and Meredith R 2001. A large-scale evaluation of amnio-PCR for the rapid prenatal diagnosis of fetal trisomy. *Ultrasound Obstet Gynecol* 17(2): 115-8.

Mann K, Donaghue C, Fox S P, Docherty Z and Ogilvie C M 2004. Strategies for the rapid diagnosis of chromosome aneuploidy. *E J Hum Genet* 12: 907-915. Mann K, Donaghue C and Ogilvie C M 2003. In vivo somatic microsatellite mutations identified in non-malignant human tissue. *Hum Genet* 114(1): 110-4.

Association for Clinical Cytogenetics and Clinical Molecular Genetics Society
QF-PCR FOR THE DIAGNOSIS OF ANEUPLOIDY BEST PRACTICE GUIDELINES (2012) V3.01

Mann K, Fox S P, Abbs S J, Yau S C, Scriven P N, Docherty Z and Ogilvie C M 2001. Development and implementation of a new rapid aneuploidy diagnostic service within the UK National Health Service and implications for the future of prenatal diagnosis. *Lancet* 358(9287): 1057-61.

Mann K, Kabba M, Donaghue C, Hills A, Ogilvie C M 2007. Analysis of a chromosomally mosaic placenta to assess the cell populations in dissociated chorionic villi: implications for QF-PCR aneuploidy testing. *Prenat Diag.* 27: 285-289.

Mansfield E S 1993. Diagnosis of Down syndrome and other aneuploidies using quantitative polymerase chain reaction and small tandem repeat polymorphisms. *Hum Mol Genet* 2(1): 43-50.

Pertl B, Kopp S, Kroisel P M, Tului L, Brambati B and Adinolfi M 1999. Rapid detection of chromosome aneuploidies by quantitative fluorescence PCR: first application on 247 chorionic villus samples. *J Med Genet* 36(4): 300-3.

Schmidt W, Jenderny J, Hecher K, Hackeloer B J, Kerber S, Kochhan L and Held K R 2000. Detection of aneuploidy in chromosomes X, Y, 13, 18 and 21 by QF-PCR in 662 selected pregnancies at risk. *Mol Hum Reprod* 6(9): 855-60.

Sullivan KM, Mannucci A, Kimpton CP, Gill P. 1993. A rapid and quantitative DNA sex test: fluorescence-based PCR analysis of XY homologous gene amelogenin. *Biotechniques* 15(4): 636-638, 640,641.

Verma L, Macdonald F, Leedham P, Mcconachie M, Dhanjal S and Hulten M 1998. Rapid and simple prenatal DNA diagnosis of Down's syndrome. *Lancet* 352(9121): 9-12.

Waters J J, Walsh S, Levett L J, Liddle S, Akinfenwa Y 2006. Complete discrepancy between abnormal fetal karyotypes predicted by QF-PCR rapid testing and karyotyped cultured cells in a first-trimester CVS. *Prenat Diag* 26(10): 829-897.

Waters J J, Mann K, Grimsley L, Ogilvie C M, Donaghue C, Staples L, Hills A, Adams T, Wilson C 2007. Complete discrepancy between QFPCR analysis of uncultured villi and karyotyping of cultured cells in the prenatal diagnosis of Trisomy 21 in three CVS. *Prenat Diagn* 27: 332-339.

11 VERSION CONTROL

issue date	Current document	summary of changes	Version replaced
20/01/2012	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2012) v3.01	Missing paragraph restored 3.4	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2012) v3.00
	Previous versions		
05/01/2012	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2012) v3.00	Amendments made to the following sections; 3.2, 5, 5.1, 6.2.1.4, 6.2.2.2, 6.2.2.3, 6.2.2.4, 7.1, 7.1.3, 8, 9.2, 9.3, 11.	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2007) v2.01
13/12/2007	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2007) v2.01	Joint ACC/CMGS identity clarified with revised logos and text on page 1, amended header (all pages)	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2007) v2.00
10/09/2007	QF-PCR for the Diagnosis of Aneuploidy Best Practice Guidelines (2007) v2.00	Reformatted to standard ACC BPG style. Significant amendments were made to Section 3 - Sample preparation and Section 8 - Mosaicism .	QF-PCR for Diagnosis of Aneuploidy (Apr 2005) (v.1.00)
April 2005	QF-PCR for Diagnosis of Aneuploidy (Apr 2005) (v1.00)		no previous version

Version 1.00 Guidelines formulated by: Kathy Mann, Caroline Ogilvie, Celia Donaghue, Roger Mountford, Ciaron McAnulty, Jon Warner, Nicola Dunlop, Lisa Levett, Carol Hardy, Cathy McConnell, Jane Diack, Fiona McKay.

Version 2.00 - guidelines updated by S. Hamilton and K. Mann in consultation with the QF-PCR email users group (meeting held 05/02/07).

Association for Clinical Cytogenetics and Clinical Molecular Genetics Society
QF-PCR FOR THE DIAGNOSIS OF ANEUPLOIDY BEST PRACTICE GUIDELINES (2012) V3.01

Circulated and approved by all ACC Heads of Department and ratified at ACC Council on 04/09/07.

Version 3.00 - guidelines updated by S. Hamilton and K. Mann.

Circulated and approved by all ACC Heads of Department and ratified by ACC Council on 10/10/11. Draft guidelines published on the CMGS website for comment and ratified at the CMGS Council on 21/11/2011.